NXT™ UNIVERSAL HEAD SUPPORT ADAPTER
ADAPTATEUR DE SUPPORT DE TÊTE
UNIVERSEL NXT™

INSTRUCTION MANUAL
MANUEL D'INSTRUCTION

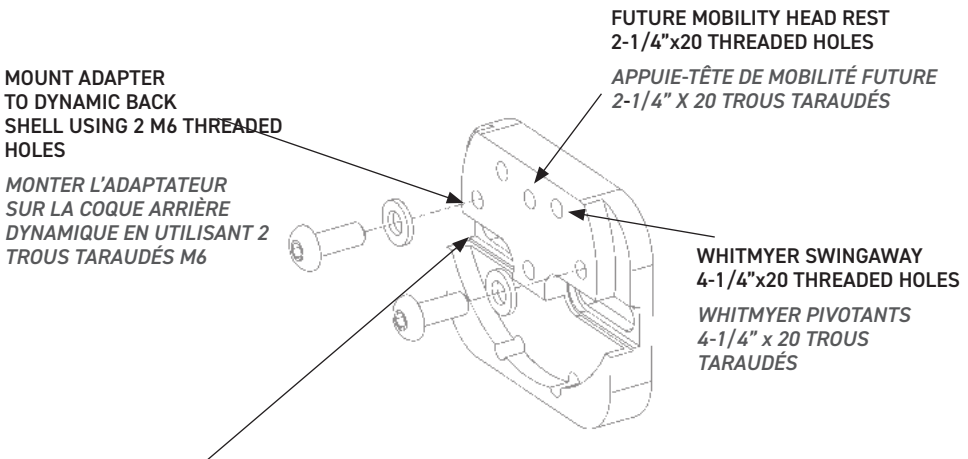
NIB-1200 UNIVERSAL HEAD SUPPORT ADAPTER

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The Universal Head Support Adapter has several threaded holes and nut slots compatible with most common head support mounting brackets as showed in the following diagram.

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SLOTS FOR M6, 1/4" OR 5/16" NUTS WITH ADJUSTABLE SPACING FOR USE WITH HEAD SUPPORT USING 2 HORIZONTALLY SPACED SCREWS SUCH AS A OTTOBOCK, INVACARE, STEALTH AND WHITMYER REMOVABLE

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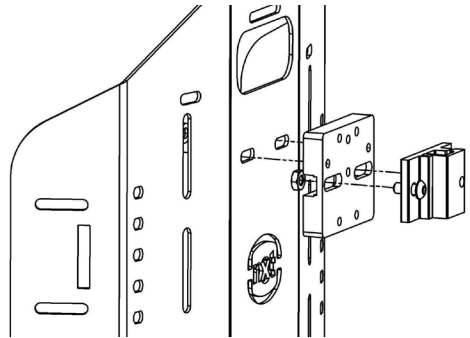
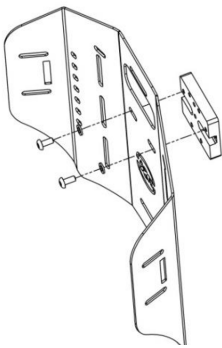
UNIVERSAL HEAD SUPPORT ADAPTER INSTALLATION

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The adapter is mounted to NXT series backs using 2 M6x14mm Button Head Cap Screws and M6 Flat Washer as shown below with an example of the installation of an OTTOBACK head support clamp.

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Universal Head Support Adapter mounted to NXT series back with M6 screws and washers.

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OTTOBACK head support clamp mounted to the adapter with 1/4"-20 screws and nuts

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The slotted holes and machined grooves allow the use of M6, 1/4" or 5/16" nuts with adjustable spacing to accommodate many different mounting arrangements. The nuts fit as shown below.

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1422791 Ontario Ltd

Dynamic Health Care Solutions

753011 Second Line EHS, Orangeville, ON L9W 5W4 Canada

519-942-8441 | www.DynamicHCS.com

Advena Limited | Tower Business Centre, 2nd Flr., Tower Street Swatar, BKR 4013 Malta